

Implanting of a REACTIV revision stem with a standard HACTIV instrument set.

A specific REACTIV revision instrument set is available for the use of such a femoral component. However the REACTIV stems can be implanted with the standard primary instrument set using round distal reamers in order to compensate difference in stem lengths. See chart below showing \emptyset of the reamer required

Diameter of required round distal reamer in order to implant the REACTIV revision stem with the standard HACTIV instrument set are shown below.

HACTIV / REACTIV

HACTIV 12 length of stem 150 mm / REACTIV 12 length of stem 180 mm
Distal round reamer \emptyset 12 mm required

HACTIV 14 length of stem 160 mm / REACTIV 14 length of stem 200 mm
Distal round reamer \emptyset 13 mm required

HACTIV 16 length of stem 170 mm / REACTIV 16 length of stem 210 mm
Distal round reamer \emptyset 13 mm required

HACTIV 18 length of stem 180 mm / REACTIV 18 length of stem 220 mm
Distal round reamer \emptyset 14 mm required

HACTIV 20 length of stem 190 mm / REACTIV 20 length of stem 240 mm
Distal round reamer \emptyset 14 mm required