

SURGICAL TECHNIQUE


*The purpose of this surgical technique is to ensure a correct instruments use and to make operating easier.
The surgeon will adapt according to his practices.*

1-Retract the anterior and posterior walls. Remove all tissues.

2-Seat the cup with the impactor (take out the apical plug and screw on the impactor). Respect the angle given by orientation guide.



3- Double inlay choice : Polyethylene or Ceramic

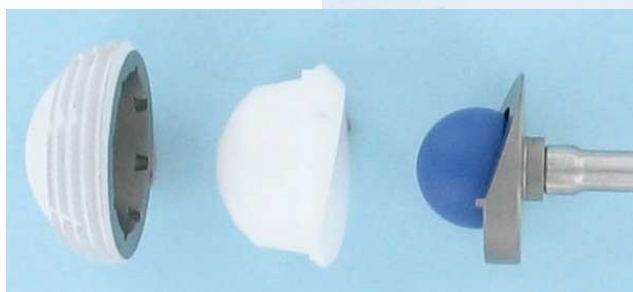

4- impact the polyethylene inlay with PE holder/ impactor. Use this instrument to orientate the inlay.

5- Or make a trial with the green trial inlays for ceramics with the insertion device. Prior to inserting the inlay, check that the inside of the cup is clean. To be able to remove the device without compromising the fixation or position of the inlay, pull the trigger to release suction vacuum.



6- Use centraliser to finalize impaction.

7-Closing